EN CIMEOSIL® SCAR AND LASER GEL

Description: CIMEOSIL® SCAR AND LASER GEL is a medical grade silicone compound (polysiloxane) that forms a very thin water-repellant membrane over the skin, like an invisible bandage.

Indications: CIMEOSIL® SCAR AND LASER GEL is indicated for the topical management of keloid and hypertrophic scars.

Directions: Use on scars after wounds have completely healed. For laser surgery patients, begin using after the skin bed is healed (usually on the 7th to 10th post-operative day). Wash the affected area and pat dry. With a clean fingertip, apply a very small amount of CIMEOSIL® GEL to the affected area. Gently wipe away any excess. Use sparingly, as excessive amounts may stain bedding or clothing. Apply twice daily. CIMEOSIL® GEL may be covered with a cosmetic. See Tube Opening Instructions at end of document.

Warnings: Care should be taken to avoid getting CIMEOSIL® GEL in the eyes or mucus membranes and when using it on newly healed tissues. Do not use on open wounds. If scar persists, redness, irritation, rash, pruritus (itchy skin), infection or maceration (whitening of skin) occurs discontinue use and consult your physician. **KEEP THIS PRODUCT OUT OF THE REACH OF CHILDREN. FOR EXTERNAL USE ONLY.**

Special Storage and/or Handling Conditions: CIMEOSIL® GEL must be stored at 16°C (60°F) to 29°C (85°F).

Disposal Instructions: This product is not biodegradable. Carefully dispose of it per local guidelines.

Product Complaints/EU Serious Incidents: If you are dissatisfied with this product or have any questions, please contact Implantech Associates, Inc. Any serious incident that has occurred in relation to the device should be reported to us and the Competent Authority of the EU Member State in which the user and/or patient is established.

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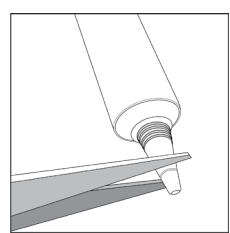
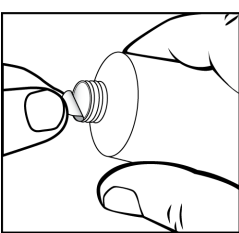
CSG-14

SYMBOLS GLOSSARY (USA only)

SYMBOL	STANDARD No.	SYMBOL TITLE & REF No.	MEANING OF SYMBOL
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII EU 2017/745 Annex V	CE marking of conformity (Class I)
	93/42/EEC EU 2017/745	CE Marking; 93/42/EEC Annex XII EU 2017/745 Annex V	CE marking of conformity (Class IIa, IIb or III)
	EN ISO 15223-1 / ISO/DIS 15223-1	Medical device; 5.7.7	Indicates the item is a medical device.
	EN ISO 15223-1	Manufacturer; 5.1.1	Indicates the medical device manufacturer.
	EN ISO 15223-1	Authorized representative in the European Community / European Union; 5.1.2	Indicates the authorized representative in the European Community / European Union.
	EN ISO 15223-1	Batch code; 5.1.5	Indicates the manufacturer’s batch code so that the batch or lot can be identified.
	EN ISO 15223-1	Catalogue number; 5.1.6	Indicates the manufacturer’s catalogue number so that the medical device can be identified.
	EN ISO 15223-1	Serial number; 5.1.7	Indicates the manufacturer’s serial number so that a specific medical device can be identified.
	EN ISO 15223-1	Date of manufacture; 5.1.3	Indicates the date when the medical device was manufactured.
	EN ISO 15223-1	Use-by-date; 5.1.4	Indicates the date after which the medical device is not to be used.
	EN ISO 15223-1	Do not re-use; 5.4.2	Indicates a medical device that is intended for one single use only.
	EN ISO 15223-1	Consult instructions for use or consult electronic instructions for use; 5.4.3	Indicates the need for the user to consult the instructions for use.
	EN ISO 15223-1 / ISO/DIS 15223-1	Caution; 5.4.4	To indicate that caution is necessary when operating the device or control close to where the symbol is placed, or to indicate that the current situation needs operator awareness or operator action in order to avoid undesirable consequences.
	EN ISO 15223-1	Do not use if package is damaged; 5.2.8	Indicates a medical device that should not be used if the package has been damaged or opened.
	EN ISO 15223-1	Temperature Limit; 5.3.7	Indicates the temperature limits to which the medical device can be safely exposed.
	ISO 15223-1	Non-sterile; 5.2.7	Indicates a medical device that has not been subjected to a sterilization process.
	EN ISO 15223-1	Sterilized using steam or dry heat; 5.2.5	Indicates a medical device that has been sterilized using steam or dry heat.
	EN ISO 15223-1	Sterilized using irradiation; 5.2.4	Indicates a medical device that has been sterilized using irradiation.
	EN ISO 15223-1 / ISO/DIS 15223-1	Double sterile barrier system; 5.2.12	Indicates two sterile barrier systems.
	EN ISO 15223-1 / ISO/DIS 15223-1	Patient identification; 5.7.3	Indicates the identification data of the patient.
	EN ISO 15223-1 / ISO/DIS 15223-1	Health care centre or doctor; 5.7.5	To indicate the address of the health care centre or doctor where medical information about the patient may be found.
	EN ISO 15223-1 / ISO/DIS 15223-1	Date; 5.7.6	To identify the date that information was entered or a medical procedure took place.
	21 CFR 801.109	Prescription Statement	"Caution: Federal USA law restricts this device to sale by or on the order of a physician".
	N/A	N/A	Quantity / Quantité / Menge / Quantità / Cantidad / Aantal / Quantidade / Mængde / Ποσότητα / Cantitate / Količina / Količina / количество / Кמות / כמות / كمية / 수량 / Мәні / Меннйiség / Magn / Quantity / Antal / Antall / Ilość / 수량 / كمية / סגור / סגור / حجم / 크기 / Stærð / Size / Storlek / Størrelse / Rozmiar / 크기 / حجم /
	N/A	N/A	Size / Taille / Größe / Dimensione / Tamaño / Formaat / Tamanho / Størrelse / Μέγεθος / Dimensione / Veličina / Velikost / Размер / Méret / Stærð / Size / Storlek / Størrelse / Rozmiar / 크기 / حجم /

EC REP

Emergo Europe

Prinsessegracht 20

2514 AP The Hague

The Netherlands

CE