EN CIMEOSIL® GEL SHEETING

Description: CIMEOSIL® GEL SHEETING is a reusable self-adhering silicone product. It is packaged together with a drying mat.

Indications: CIMEOSIL® GEL SHEETING may be used for the management of keloid and hypertrophic scars. The device is not intended to be used on open wounds.

Directions: Gently clean the scar area with mild soap and water and pat dry. Remove sheeting from plastic bag and cut to a size slightly larger than the scar to ensure it is fully covered. Remove protective plastic film from sticky side of sheeting and apply to affected area for about 8 hours daily. Depending on scar age and depth, scar should begin to become less visible within 7 to 90 days. Use until desired results are obtained. When necessary, wash sheeting with mild soap, thoroughly rinse with water, then dry completely prior to reuse. Place sticky side of sheeting on drying mat, dry, then store sheeting in sealable pouch when not in use. See Drying Mat Instructions at end of document.

Warnings: Do not use on open wounds. If scar persists, redness, irritation, rash, pruritus (itchy skin), infection or maceration (whitening of skin) occurs discontinues use and consult your physician. Do not use talcum powder, skin moisturizers, lotions, sunscreen or any topical medications on area to be treated while using sheeting. Apply new sheet when current sheet is no longer adherent.

Special Storage and/or Handling Conditions: Sheeteting must be stored at 16°C (60°F) to 29°C (85°F).

Disposal Instructions: This product is not biodegradable. Carefully dispose of it per local guidelines.

Product Complaints/EU Serious Incidents: If you are dissatisfied with this product or have any questions, please contact Implantech Associates, Inc. Any serious incident that has occurred in relation to the device should be reported to us and the Competent Authority of the EU Member State in which the user and/or patient is established.

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